

The OmniPod DASH

Basic Information:

- The Omnipod is a small, tubeless, wearable, waterproof device. It holds up to 200units (2mL) of insulin and sticks directly onto the body (unlike the other two insulin pumps that use tubing).
- This Pod communicates wirelessly with the “Personal Diabetes Manager” (PDM) device that looks like a cell phone. This device is what contains all the pumps information & allows a person to adjust settings.
- The Pod gets replaced every 3 days with a new one, the PDM has a rechargeable battery.



The “PDM”
(Can also use a phone)



The “Pod”
(Filled with insulin & tubeless)

1. HOW DO I UNLOCK THE SCREEN?

Press the “home” button on the left side of the key pad.

2. HOW DO I FIND THE PUMP SETTINGS?

Use the arrow buttons to move up and down. To select, use the top right soft key button under the word “select”.

Basal Settings:

Home → Settings → Basal programs → Basal → View

Bolus Settings:

Home → Settings → System setup → “bolus/basal/calcs” → “ratios/factors/targets” → review all settings

3. HOW DO I SUSPEND INSULIN DELIVERY?

Home → Suspend

